5.6 Lx1: 105-048455-00

Instructions for Use

Hemoglobin A1c Hemolysis Solution

Find IFUs in more languages on our website:
<https://www.mindray.com/en/resources-center/ivd-edocs>

COMPANY CONTACT / KONTAKT MIT DEM UNTERNEHMEN/CONTACTO CON LA EMPRESA / COORDONNEES DE LA SOCIETE/INFORMAZIONI RELATIVE ALLA SOCIETÀ / ŞİRKET İLETİŞİM / CONTATO DA EMPRESA / КОНТАКТНАЯ ИНФОРМАЦИЯ

	Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
	Mindray Building, Keji 12 th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China
	www.mindray.com
	service@mindray.com
Tel /Tél/Тел:	+86 755 81888998
Fax/Télécopie/ Faks/Факс:	+86 755 26582680

	Shanghai International Holding Corp. GmbH(Europe)
	Eiffestraße 80, Hamburg 20537, Germany
Tel /Tél/Тел:	0049-40-2513175
Fax/Télécopie/ Faks/Факс:	0049-40-255726

Revision bar indicates update to previous version / Das Revisionsbarsymbol weist auf eine Aktualisierung der vorigen Version / La barra de revisión indica una actualización a la versión anterior / La barre de révision indique la mise à jour vers la version précédente / La barra di revisione indica l'aggiornamento alla versione precedente / Değişiklik çubuğu, önceki sürüme yapılan güncellemeyi gösterir / A barra de revisão indica a atualização para a versão anterior / Строка обновления указывает на обновление предыдущей версии

Hemoglobin A1c Hemolysis Solution

Packing Specifications

Table 1. Packing specification

Model	Packing specification
H-12H	5.6 L×1

Intended Use

This product is used with Mindray H-120/H-120P Automated Glycohemoglobin Analyzer. It is used for pre-treatment of blood samples before Hemoglobin A1c (HbA1c) analysis.

Test Principle

Being mixed with whole blood samples, the product pre-processes the samples for further analysis by breaking the cell membranes to release hemoglobin from the erythrocytes.

Major Components

Polyoxyethylene non-ionic surfactants.....≤ 0.16%
 Phosphate buffer.....≤ 0.3%

Storage Conditions and Stability

- The kit may be used up to the expiry date indicated on the label if stored unopened at 2 to 30°C, in a well-ventilated area, away from corrosive gas and direct sunlight.
- The working temperature range of the product is consistent with that of its applicable instruments. The open-vial validity is 100 days.
- For the production date and expiry date, see the package or the label.

Applicable Instruments

This product applies to H-120 and H-120P Automated Glycohemoglobin Analyzers manufactured by Shenzhen Mindray Bio-Medical Electronics Co., Ltd..

Material Required but not Provided

The following materials are required but not provided with the product: Mindray manufactured measurement instruments and matched reagents, as well as general laboratory devices.

Sample Collection and Preparation

- Fresh human venous whole blood samples anticoagulated with EDTA K₂, EDTA K₃, lithium heparin, sodium citrate, potassium oxalate or sodium fluoride.
- For more information regarding the requirements for sample collection and preparation, refer to the Operator's Manual of the applicable instruments.

Reagent Preparation

The product is a ready-for-use reagent.

Test Procedures

1. Restore the product to usage temperature.
2. Open the external packing of the product, and connect the product container to the instrument.
3. Follow the instructions in the Operator's Manual of the applicable instrument to replace the reagent in the instrument, and dispose the old reagent container properly.

For detailed information, refer to the Operator's Manual of the applicable instrument.

Cut-off Value or Reference Intervals

N/A

Result Elaboration

N/A

Limitation

N/A

Performance Specifications

Appearance

The product is transparent liquid without deposits, suspended grains or flocks.

Warnings and Precautions

- For in-vitro diagnostic use only. For laboratory professional use.
- Read the instruction for use carefully before use, and use the product according to the instructions.
- The product is transparent liquid without deposits, suspended grains or flocks. If it gets polluted or affected by other factors and becomes

abnormal, stop using it and replace it with a normal one.

- Do not mix reagents of different batches.
- If the reagent is frozen, thaw the frozen reagent at room temperature and ensure it is thoroughly mixed before using it.
- The sample analysis may be impacted by various factors, including the use of expired or ineffective reagents, mixing reagents with third-party products, combining residue from an old container with newly-opened reagents, and using reagents outside of their specified conditions.
- Do not take the product into mouth. If you accidentally ingest the product, seek medical treatment immediately.
- Avoid exposure to skin. If you accidentally spill the product on your skin, wash it off with plenty of water immediately.
- Avoid exposure to eyes. If you accidentally spill the product into your eyes, wash it off with plenty of water immediately, and seek medical treatment if necessary.
- Confirm the integrity of the package before use. Do not use the reagents with damaged packages. The results cannot be assured when the reagents are stored at inappropriate conditions.
- If the reagents are opened unintentionally before use, they shall be used as soon as possible.
- Disposal of waste liquid and materials should be in accordance with local guidelines.
- The Material Safety Data Sheet (SDS) is available upon request.
- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the country in which the user and/or the patient is established.

References

N/A

Symbols

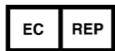
Batch code

Use-by date

Temperature limit

Authorized
representative in the
European Community

Manufacturer

*In vitro* diagnostic
medical deviceConsult instructions
for useKeep away
from sunlight
(Indicates a medical
device that needs
protection from
light source)

Catalogue number

European
ConformityUnique device
identifier

Approval Date of the Instructions for Use

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