T4

Total Thyroxine (CLIA)

INSTRUCTIONS FOR USE

mindray

T4

Total Thyroxine (CLIA)

Order Information

Catalog No.	Package Size
105-004211 -00	2×50 tests
105-004238 -00	2×100 tests

Intended Use

The CL-series T4 assay is a Chemiluminescent Immunoassay (CLIA) for the quantitative determination of total thyroxine (T4) levels in human serum or plasma.

Measurement obtained by this device is used as an aid in the diagnosis and treatment of thyroid disease.

Summary

Thyroxine (T4) is a 777 daltons iodine-containing hormone secreted by the thyroid gland. T4 is 99.9% bound to serum thyroxine binding proteins (TBP). T4 and T3 regulate normal growth and development. Although T3 has greater biologic potency, T4 is about 50-fold excess of circulating T3¹ and combines more than 90% of the circulating protein-bound iodine. Thyroxine is commonly measured as total T4. In most patients the normal T4 level indicates good thyroid status, however, T4 level can be affected by the change in binding proteins while the level of unbound hormone unchanged.² Drugs that compete for protein binding sites, such as phenylbutazone, diphenylhydantoin or salicylates, can result in a depressed T4 measurement.³⁻⁵ Thus, the final definition of thyroid status should be determined in conjunction with other thyroid function tests such as TSH, FT4, T3, FT3 and clinical evaluation.

Assay Principle

The CL-series T4 assay is a competitive binding immunoenzymatic assay to determine the level of total T4.

In the first step, sample, stripping agent are added into a reaction cuvette. During incubation, the stripping agent dissociate T4 from the binding proteins.

In the second step, paramagnetic microparticle coated with monoclonal anti-T4 antibody (mouse) and T4 alkaline phosphatase conjugate are added into a reaction cuvette. T4 in the sample competes with the T4 alkaline phosphatase conjugate. After incubation, the T4 alkaline phosphatase binds to T4 antibody coated microparticle to form a sandwich complex. Microparticle is magnetically captured while other unbound substances are

removed by washing.

In the third step, the substrate solution is added to the reaction cuvette. It is catalyzed by T4 alkaline phosphatase conjugate in the immunocomplex retained on the microparticle. The resulting chemiluminescent reaction is measured as relative light units (RLUs) by a photomultiplier built into the system. The amount of total T4 present in the sample is inversely proportional to RLUs generated during the reaction. The total T4 concentration can be determined via a calibration curve.

Reagent Components

Ra	Paramagnetic microparticle coated with monoclonal anti-T4 antibody (mouse). Minimum concentration: 0.1 g/L. TRIS ^{a)} buffer: 50 mmol/L. Preservatives: 0.05% ProClin 300 and 0.09% sodium azide.
Rb	T4 alkaline phosphatase conjugate. Minimum concentration: 0.1 mg/L. MES ^{b)} buffer: 50 mmol/L. Preservatives: 0.05% ProClin 300 and 0.09% sodium azide.
Rc	8-Anilino-1-Napthalenesulfonic Acid (ANS). Minimum concentration: 2 g/L. TRIS buffer: 50 mmol/L. Preservatives: 0.05% ProClin 300 and 0.09% sodium azide.

a) TRIS=Tris (hydroxymethyl)-aminomethane

b) MES=2-(N-morpholino)ethanesulfonic acid

Storage and Stability

The unopened T4 (CLIA) reagent kit is stable up to the stated expiration date when stored at 2-8°C.

The T4 (CLIA) reagent kit can be stored onboard and used for a maximum of 56 days after opening at 2-8°C.

Reagent Preparation

The reagents in the kit have been assembled into an intact unit that cannot be separated.

Materials Required but not Provided

Mindray CL-series Chemiluminescence Immunoassay Analyzer
Cat.No.105-004280-00: Total T4 Calibrators, 1×2.0 mL for each of calibrator C0, C1 and C2.

Cat.No.105-007383-00: Thyroid Function Multi Control (L), 1×5.0 mL

Cat.No.105-007384-00: Thyroid Function Multi Control (H), 1×5.0 mL

Cat.No.105-007379-00: Thyroid Function Multi Control (L), 3×5.0 mL

Cat.No.105-007380-00: Thyroid Function Multi Control (H), 3×5.0 mL

Cat.No.105-007371-00: Thyroid Function Multi Control (L), 6×5.0 mL

Cat.No.105-007372-00: Thyroid Function Multi Control (H), 6×5.0 mL

Cat.No.105-007375-00: Thyroid Function Multi Control (L), 12×5.0 mL

Cat.No.105-007376-00: Thyroid Function Multi Control (H), 12×5.0 mL

Cat.No.105-004552-00: Wash Buffer, 1×10 L

Cat.No.105-009044-00: Substrate Solution, 4×75 mL

Cat.No.105-004274-00: Substrate Solution, 4×115mL

Reaction Cuvette.

Applicable Instrument

Mindray CL-series Chemiluminescence Immunoassay Analyzer

Specimen Collection and Preparation

Specimen Types

- Human serum or plasma collected in K₂EDTA, sodium heparin and lithium heparin are recommended for this assay.
- Blood collection tubes from various manufacturers may contain additives which could affect the test results in some cases. Not all available tubes on the market were tested by Mindray. Each laboratory should determine the acceptability of different blood collection tubes and serum/plasma separation products. Variations in these products may exist among manufacturers and from lot-to-lot.
- When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Specimen Conditions

- Do not use:
 - heat-inactivated specimens
 - grossly hemolyzed specimens
 - specimens with obvious microbial contamination

- For accurate results, serum and plasma specimens should be free of fibrin, red blood cells, and other particulate matter. Serum specimens from patients receiving anticoagulant or thrombolytic therapy may contain fibrin due to incomplete clot formation.

Preparation for Analysis

- Follow blood collection tube manufacturer's recommendations for centrifugation. Centrifuge the specimens after clot formation is complete. Ensure residual fibrin and cellular matter has been removed prior to analysis.
- For optimal results, inspect all samples for bubbles. Remove bubbles with a pipette tip prior to analysis. Specimens must be mixed thoroughly after thawing. Thawed samples should be centrifuged prior to use.
- If the sample was covered with lipid layer after centrifugation, the sample should be transferred to a clean tube and centrifuged before testing. Do not transfer the lipid layer. Handle carefully to prevent cross contamination.

Specimen Storage

- Specimens should be tested timely after sample collection. If the assay cannot be completed within 8 hours, refrigerate the samples at 2 to 8°C. If testing will be delayed for more than 72 hours, specimens should be frozen at -20°C or colder. The specimens can be stored at -20°C for up to 30 days.
- The number of repeated freeze and thaw cycles shall not exceed 5 times.

Assay Procedure

For optimal performance of this assay, operators should read the related system operation manual carefully, to get sufficient information such as operation instructions, sample preservation and management, safety precaution, and maintenance. Prepare all required materials for the assay as well.

Before loading the T4 (CLIA) reagent kit on the machine for the first time, unopened reagent bottle should be inverted gently for at least 30 times to resuspend the microparticles that have settled during shipment or storage. Visually inspect the bottle to ensure the microparticles have been resuspended. If the microparticles remain adhered to the bottle, continue inverting until the microparticles have been completely resuspended. If the microparticles cannot be resuspended, it is recommended not to use this bottle of reagent. Contact Mindray Customer Service for help. Do not invert opened reagent bottle.

This assay requires 20 μ L of sample for a single test. This volume does not include the dead volume of the sample container.

Additional volume is required when performing additional tests from the same sample. Operators should refer to the system operation manual and specific requirement of the assay to determine the minimum sample volume.

Calibration

CL-series T4 (CLIA) has been standardized against the IDMS (isotope dilution mass spectrometry) reference measurement procedure.

The specific information of master calibration curve of T4 (CLIA) reagent kit is stored in the two-dimensional barcode attached in the reagent pack. It's used together with calibrators for the calibration of the specific reagent lot. When performing calibration, scan the information of master calibration curve from the barcode into the system first, and then use the calibrators at three levels. Valid calibration curve is required before any T4 test. Recalibration is recommended every 4 weeks, or when a new reagent lot is used, or the quality controls are out of specified range. For detailed instruction of calibration, refer to the system operation manual.

Quality Control

It is recommended to run quality controls once every 24 hours if the tests are in use, or after every calibration. The frequency of using quality control should be adapted to each laboratory's requirements. The recommended two levels of quality controls for this assay are Mindray Thyroid Function Multi Control (L) and Thyroid Function Multi Control (H). In addition, other suitable control material can be used.

Quality control results should be within the acceptable ranges. If a control is out of its specified range, the associated test results are invalid and the samples must be retested. Recalibration may be required. Examine the assay system referring to the system operation manual. If the quality control results are still out of the specified range, please contact Mindray Customer Service for help.

Calculation

The analyzer automatically calculates the analyte concentration of each sample based on the master calibration curve read from the barcode, and the RLUs generated from the three level calibrators of defined concentration values. The results are shown in the unit of $\mu\text{g/dL}$ or nmol/L (selectable).

Conversion factors: $\mu\text{g/dL} \times 12.87 = \text{nmol/L}$

$\text{nmol/L} \times 0.077688 = \mu\text{g/dL}$

Dilution

Samples with T4 concentrations above the upper limit can be diluted with sample diluent. The recommended dilution is 1:2

(either automatically by the analyzer or manually). The concentration of the diluted sample must be $> 15 \mu\text{g/dL}$. If use manual dilution, multiply the result by the dilution factor. After automated dilution by the analyzers, the system automatically multiply the result by the dilution factor when calculating the sample concentration.

Expected values

A normal range of $5.0 \mu\text{g/dL}$ to $14.5 \mu\text{g/dL}$ (central 95% interval) was obtained by testing serum specimens from 531 non-pregnant individuals determined as normal by MINDRAY TSH and Free T4 assays.

Representative T4 ranges during normal pregnancy are summarized below.

Pregnancy Gestation	N	Central 95% Range
First Trimester	154	$7.0 - 16.4 \mu\text{g/dL}$
Second Trimester	158	$7.7 - 17.6 \mu\text{g/dL}$
Third Trimester	152	$6.6 - 16.9 \mu\text{g/dL}$

It is highly recommended that each laboratory establishes its own normal range, which may be unique to the population it serves depending upon geography, race, sex, age, disease, dietary, or environmental factors.

Limitation

The upper limit of this assay is $30 \mu\text{g/dL}$. A specimen with a total T4 concentration lower than the upper limit can be quantitatively determined, while specimen with a concentration higher than the upper limit will be reported as $> 30 \mu\text{g/dL}$ or retested by diluting the samples with Mindray sample diluent.

The concentration of total T4 in a given specimen, determined with assays from different manufacturers, can vary due to differences in assay methods, calibration, and reagent specificity. The assay results should be used in conjunction with other data, such as symptoms, results of other tests, clinical history, etc.

Specimens of hyperlipidemia, icterus or hemolysis may result in incorrect results.

Heterophilic antibodies in samples can react with immunoglobulin in kit and interfere the immunoassay results.⁶ Patients who have been regularly exposed to animals or have been treated with immunoglobulins or immunoglobulin fragments may produce such antibodies, resulting in abnormal results. For example, specimens from individuals who have been exposed to mouse monoclonal antibodies may contain human anti-mouse antibodies (HAMA). Such specimens may show either falsely elevated or depressed values with assay kits employing mouse monoclonal

antibodies.^{7,8} There are anti-interference components in Mindray T4 (CLIA) reagent kit, which can effectively reduce the influence caused by HAMA in samples, but there may still be interference problems in a few samples. Clinical examinations, medical history, or other relevant information are needed to determine the diagnosis of the patient.

Performance Characteristics

Lower limits of measurement

Limit of Blank, Limit of Detection and Limit of Quantitation

Limit of Blank = 0.4 µg/dL

Limit of Detection = 0.5 µg/dL

Limit of Quantitation = 1.3 µg/dL

The Limit of Blank, Limit of Detection and Limit of Quantitation were determined in accordance with the CLSI (Clinical and Laboratory Standards Institute) EP17-A2 requirements.

The Limit of Blank is the 95th percentile value from $n \geq 60$ measurements of analyte-free samples over several independent series. The Limit of Blank corresponds to the concentration below which analyte-free samples are found with a probability of 95 %.

The Limit of Detection is determined based on the Limit of Blank and the standard deviation of low concentration samples. The Limit of Detection corresponds to the lowest analyte concentration which can be detected (value above the Limit of Blank with a probability of 95 %).

The Limit of Quantitation is the lowest analyte concentration that can be reproducibly measured with an intermediate precision CV of ≤ 20 %.

Measuring Range

0.5-30 µg/dL (defined by the Limit of Detection and the maximum of the master calibration curve). Values below the Limit of Detection are reported as < 0.5 µg/dL. Values above the measuring range are reported as > 30 µg/dL (or up to 60 µg/dL for 2-fold diluted samples).

Analytical Specificity

Hemoglobin up to 500 mg/dL, bilirubin up to 10 mg/dL, triglycerides up to 1800 mg/dL, rheumatoid factor up to 100 IU/mL or antinuclear antibody will not interfere with the Mindray CL-series T4 assay. Criterion: recovery within ± 10 % of initial value.

In vitro tests were performed on 24 commonly used pharmaceuticals. No interference with the assay was found.

No interference was observed from the following substances at

the levels indicated below. Criterion: recovery within $\pm 10\%$ of initial value.

Test Compound	Interference Substance Concentration
Methimazol	1 mg/dL
Thiouracil	5.0 mg/dL
Phenylbutazone	5.0 mg/dL

No obvious cross reactivity was observed when Mindray Total T4 Calibrator C0 was spiked with the T4 analogue at specific levels indicated in the table below. Criterion: Cross reactivity $\leq 3\%$ (Reverse T3: Cross reactivity $\leq 10\%$). The results are summarized in the table below*.

T4 Analogue	Cross-reactant Concentration	Cross Reactivity
T3	500 $\mu\text{g/dL}$	2.0%
Reverse T3	100 $\mu\text{g/dL}$	0.0%
Monoiodotyrosine	5000 $\mu\text{g/dL}$	0.0%
Diiodo-L-tyrosine	5000 $\mu\text{g/dL}$	0.0%

*Representative data; results in individual laboratories may vary.

Accuracy

Two trueness controls with traceable and defined values were used to verify the accuracy of this assay. The results showed that the relative deviation was within $\pm 10\%$. The results are listed in the following table*.

Sample	Measured T4 Value ($\mu\text{g/dL}$)	Defined T4 Value ($\mu\text{g/dL}$)	Relative Deviation
Level 1	10.01	9.54	4.96%
Level 2	15.65	15.62	0.17%

*Representative data; results in individual laboratories may vary.

Precision

Precision was determined by following Clinical and Laboratory Standards Institute (CLSI) Protocol EP5-A2⁹. Two levels of quality controls were tested in duplicate in two separate runs per day, for a total of 20 days, using a single lot of reagents and a single calibration curve. The precision data are summarized in the table below*.

Sample	Mean T4 ($\mu\text{g/dL}$)	Within-run CV	Between-run CV	Within-Device CV
1	9.90	1.77%	1.74%	4.59%
2	16.09	1.77%	2.40%	4.93%

Linearity

A high concentration T4 sample (approximately 30 µg/dL) was mixed with a low concentration sample (< 0.5 µg/dL) at different ratios, generating a series of dilutions. The total T4 of each dilution was determined using the Mindray CL-Series T4 Assay. Linearity was demonstrated in the range of 0.5 µg/dL to 30 µg/dL, the correlation coefficient r is ≥ 0.990 . The linearity data are summarized in the table below*.

Concentration (µg/dL)	1	2	3	4	5	6
Expected T4	0.00	6.70	13.41	20.11	26.82	33.52
Measured T4	0.00	6.11	11.98	17.99	24.87	33.52

Method Comparison

The Mindray CL-Series T4 Assay was compared to a commercially available diagnostic kit in a correlation study with about 306 specimens. The statistical data obtained by linear regression are shown in the table below.

Concentration Range (µg/dL)	Slope	Intercept	Correlation Coefficient
0.5-30	1.0191	0.6585	0.986

Warnings and Precautions

- For in vitro diagnostic use only. For laboratory professional use.
- Follow all the rules in handling laboratory reagents and take necessary safety precautions.
- The concentration of total T4 in a given specimen determined with different manufacturers can vary due to differences in assay methods and reagent specificity. The results reported by the laboratory to the physician must include the identity of the T4 assay used. Values obtained with different assay methods cannot be used interchangeably.
- Do not use reagent kits beyond the expiration date.
- Do not use reagents mixed from different reagent lots.
- Always keep the reagent pack in the upright position to ensure no microparticle has been lost prior to use.
- Reagent pack opened for more than 56 days is not recommended for use.
- Reliability of assay results cannot be guaranteed if the instructions in this package insert are not followed.
- All the specimen and reaction wastes should be considered

potentially biohazard. The handling of specimens and reaction wastes should be in accordance with the local regulations and guidelines.

10. The Material Safety Data Sheet (MSDS) is available upon request.
11. Please confirm the integrity of the package before use. Do not use the reagents with damaged packages.
12. If the reagents are opened unintentionally before use, they shall be used as soon as possible.
13. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the local competent authority.
14. Instability or deterioration should be suspected if there are visible signs of leakage, turbidity, precipitates or microbial growth.
15. Do not freeze. The results cannot be assured when the reagents are stored at inappropriate condition.
16. This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

**Warning**

H317 May cause an allergic skin reaction.

H412 Harmful to aquatic life with long lasting effects.

Prevention:

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P272 Contaminated work clothing should not be allowed out of the workplace.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P273 Avoid release to the environment.

Response:

P302 + P352 IF ON SKIN: Wash with plenty of water.

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.

P362 + P364 Take off contaminated clothing and wash it before reuse.

Disposal:

P501 Dispose of contents/ container in accordance with local regulation.

Graphical Symbols

In vitro
diagnostic
medical
device

Authorized
representative
in the European
Community

Consult
instructions
for use

European
Conformity

Catalogue
number

Temperature
limit

Manufacturer

Use-by
date

Caution

Batch code

This way up

Unique
device
identifier

References

1. Lerman J. The Physiologic Activity of L-Triiodothyronine. J Clin Endocrinol Metab 1953; 13: 1341-1346.
2. White, GH, Recent Advances in Routine Thyroid Function Testing, CRC – Critical Reviews in Clinical Laboratory.
3. Witherspoon LR, Shuler SE. Estimation of Free Thyroxine Concentration: Clinical Methods and Pitfalls. J Clin Immunoassay 1984; 7: 192-205.
4. Bermudez F, Surks MI, Oppenheimer JH. High Incidence of Decreased Serum Triiodothyronine Concentration in Patients with Nonthyroid Disease. J Clin Endocrinol Metab 1975; 41: 27-40
5. Larsen PR. Triiodothyronine: Review of Recent Studies of Its Physiology and Pathophysiology in Man. Metabolism 1972; 21: 1073-1092
6. Boscatto Lm, Stuart MC. Heterophilic Antibodies: a Problem for All Immunoassays. Clin Chem 1988; 34: 27-33.
7. Kricka L. Interferences in Immunoassays - Still a Threat.

Clin Chem 2000; 46: 1037-1038.

8. Bjerner J, et al. Immunometric Assay Interference: Incidence and Prevention. Clin Chem 2002; 48: 613-621.
9. CLSI. EP5-A2: Vol. 24, No. 25, Evaluation of Precision Performance of Quantitative Measurement Method; Approved Guideline- Second Edition.

© 2013-2023 Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

All rights Reserved

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray Building, Keji 12th Road South, High-tech Industrial Park, Nanshan, Shenzhen, 518057 P.R.China

E-mail Address: service@mindray.com

Website: www.mindray.com

Tel: +86-755-81888998

Fax: +86-755-26582680

EC-Representative: Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestraße 80, Hamburg 20537, Germany

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1. Lerman J. The Physiologic Activity of L-Triiodothyronine. *J Clin Endocrinol Metab* 1953; 13: 1341-1346.
2. White, GH, Recent Advances in Routine Thyroid Function Testing, *CRC – Critical Reviews in Clinical Laboratory*.
3. Witherspoon LR, Shuler SE. Estimation of Free Thyroxine Concentration: Clinical Methods and Pitfalls. *J Clin Immunoassay* 1984; 7: 192-205.
4. Bermudez F, Surks MI, Oppenheimer JH. High Incidence of Decreased Serum Triiodothyronine Concentration in Patients with Nonthyroid Disease. *J Clin Endocrinol Metab* 1975; 41: 27-40
5. Larsen PR. Triiodothyronine: Review of Recent Studies of Its Physiology and Pathophysiology in Man. *Metabolism* 1972; 21: 1073-1092
6. Boscatto Lm, Stuart MC. Heterophilic Antibodies: a Problem for All Immunoassays. *Clin Chem* 1988; 34: 27-33.

7. Kricka L. Interferences in Immunoassays - Still a Threat. Clin Chem 2000; 46: 1037-1038.
8. Bjerner J, et al. Immunometric Assay Interference: Incidence and Prevention. Clin Chem 2002; 48: 613-621.
9. CLSI. EP5-A2: Vol. 24, No. 25, Evaluation of Precision Performance of Quantitative Measurement Method; Approved Guideline – Second Edition.

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Exclusivo
in vitro
do dispositivo

Referências

1. Lerman J. The Physiologic Activity of L-Triiodothyronine. *J Clin Endocrinol Metab* 1953; 13: 1341-1346.
2. White, GH, Recent Advances in Routine Thyroid Function Testing, *CRC – Critical Reviews in Clinical Laboratory*.
3. Witherspoon LR, Shuler SE. Estimation of Free Thyroxine Concentration: Clinical Methods and Pitfalls. *J Clin Immunoassay* 1984; 7: 192-205.
4. Bermudez F, Surks MI, Oppenheimer JH. High Incidence of Decreased Serum Triiodothyronine Concentration in Patients with Nonthyroid Disease. *J Clin Endocrinol Metab* 1975; 41: 27-40
5. Larsen PR. Triiodothyronine: Review of Recent Studies of Its Physiology and Pathophysiology in Man. *Metabolism* 1972; 21: 1073-1092
6. Boscatto Lm, Stuart MC. Heterophilic Antibodies: a Problem for All Immunoassays. *Clin Chem* 1988; 34: 27-33.
7. Kricka L. Interferences in Immunoassays - Still a Threat. *Clin Chem* 2000; 46: 1037-1038.
8. Bjerner J, et al. Immunometric Assay Interference: Incidence and Prevention. *Clin Chem* 2002; 48: 613-621.
9. CLSI. EP5-A2: Vol. 24, No. 25, Evaluation of Precision Performance of Quantitative Measurement Method; Approved Guideline – Second Edition.

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Fabricante: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Endereço: Mindray Building, Keji 12th Road South, High-tech Industrial Park, Nanshan, Shenzhen, 518057 P.R.China

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Referencias

1. Lerman J. The Physiologic Activity of L-Triiodothyronine. J Clin Endocrinol Metab 1953; 13: 1341-1346.
2. White, GH, Recent Advances in Routine Thyroid Function Testing, CRC – Critical Reviews in Clinical Laboratory.
3. Witherspoon LR, Shuler SE. Estimation of Free Thyroxine Concentration: Clinical Methods and Pitfalls. J Clin Immunoassay 1984; 7: 192-205.
4. Bermudez F, Surks MI, Oppenheimer JH. High Incidence of Decreased Serum Triiodothyronine Concentration in Patients with Nonthyroid Disease. J Clin Endocrinol Metab 1975; 41: 27-40
5. Larsen PR. Triiodothyronine: Review of Recent Studies of Its Physiology and Pathophysiology in Man. Metabolism 1972; 21: 1073-1092
6. Boscato Lm, Stuart MC. Heterophilic Antibodies: a Problem for All Immunoassays. Clin Chem 1988; 34: 27-33.
7. Kricka L. Interferences in Immunoassays - Still a Threat. Clin Chem 2000; 46: 1037-1038.
8. Bjerner J, et al. Immunometric Assay Interference: Incidence and Prevention. Clin Chem 2002; 48: 613-621.
9. CLSI. EP5-A2: Vol. 24, No. 25, Evaluation of Precision Performance of Quantitative Measurement Method; Approved Guideline – Second Edition.

© 2013-2023 Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Todos los derechos reservados

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Riferimenti

1. Lerman J. The Physiologic Activity of L-Triiodothyronine. J Clin Endocrinol Metab 1953; 13: 1341-1346.
2. White, GH, Recent Advances in Routine Thyroid Function Testing, CRC – Critical Reviews in Clinical Laboratory.
3. Witherspoon LR, Shuler SE. Estimation of Free Thyroxine Concentration: Clinical Methods and Pitfalls. J Clin Immunoassay 1984; 7: 192-205.
4. Bermudez F, Surks MI, Oppenheimer JH. High Incidence of Decreased Serum Triiodothyronine Concentration in Patients with Nonthyroid Disease. J Clin Endocrinol Metab 1975; 41: 27-40
5. Larsen PR. Triiodothyronine: Review of Recent Studies of Its Physiology and Pathophysiology in Man. Metabolism 1972; 21: 1073-1092
6. Boscato Lm, Stuart MC. Heterophilic Antibodies: a Problem for All Immunoassays. Clin Chem 1988; 34: 27-33.
7. Kricka L. Interferences in Immunoassays - Still a Threat. Clin Chem 2000; 46: 1037-1038.
8. Bjerner J, et al. Immunometric Assay Interference:

Incidence and Prevention. Clin Chem 2002; 48: 613-621.

9. CLSI. EP5-A2: Vol. 24, No. 25, Evaluation of Precision Performance of Quantitative Measurement Method; Approved Guideline – Second Edition.

© 2013-2023 Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Tutti i diritti riservati

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Indirizzo: Mindray Building, Keji 12th Road South, High-tech Industrial Park, Nanshan, Shenzhen, 518057 P.R.China

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Haut

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Références

1. Lerman J. The Physiologic Activity of L-Triiodothyronine. J Clin Endocrinol Metab 1953; 13: 1341-1346.
2. White, GH, Recent Advances in Routine Thyroid Function Testing, CRC – Critical Reviews in Clinical Laboratory.
3. Witherspoon LR, Shuler SE. Estimation of Free Thyroxine Concentration: Clinical Methods and Pitfalls. J Clin Immunoassay 1984; 7: 192-205.
4. Bermudez F, Surks MI, Oppenheimer JH. High Incidence of Decreased Serum Triiodothyronine Concentration in Patients with Nonthyroid Disease. J Clin Endocrinol Metab 1975; 41: 27-40.

5. Larsen PR. Triiodothyronine: Review of Recent Studies of Its Physiology and Pathophysiology in Man. *Metabolism* 1972; 21: 1073-1092.
6. Boscatto Lm, Stuart MC. Heterophilic Antibodies: a Problem for All Immunoassays. *Clin Chem* 1988; 34: 27-33.
7. Kricka L. Interferences in Immunoassays - Still a Threat. *Clin Chem* 2000; 46: 1037-1038.
8. Bjerner J, et al. Immunometric Assay Interference: Incidence and Prevention. *Clin Chem* 2002; 48: 613-621.
9. CLSI. EP5-A2: Vol. 24, No. 25, Evaluation of Precision Performance of Quantitative Measurement Method; Approved Guideline – Second Edition.

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Fabricant : Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Représentant en Europe : Shanghai International Holding Corp. GmbH (Europe)