Anti-Tg Calibrators

Order Information

Catalog No.	Package Size
Anti-Tg211	C0:1×2.0 mL/vial C1:1×2.0 mL/vial C2:1×2.0 mL/vial
Anti-Tg212	C0:1×1.2 mL/vial C1:1×1.0 mL/vial C2:1×1.0 mL/vial

Intended Use

Mindray Anti-Tg Calibrators (Anti-Tg CAL) are intended to calibrate the quantitative antibody to thyroglobulin (Anti-Tg) assay on Mindray CL-series Chemiluminescence Immunoassay Analyzer.

Summary

Mindray CL measurement system is composed of Mindray CL-series Chemiluminescence Immunoassay Analyzer, Mindray reagent kits, calibrators, and controls. Anti-Tg Calibrators contain antibody to thyroglobulin (Anti-Tg). The mathematical relationship between the measured chemiluminescence light emission and the known analyte concentration establishes the calibration curve. It is used to convert Relative Light Units (RLUs) measured from samples to quantitative analyte concentration.

Components

C0,C1,C2	Antibody to thyroglobulin in buffered bovine serum albumin (BSA) at three concentration levels
----------	--

Warnings and Precautions

- For in vitro diagnostic use only.
- Professionals must be trained to take all necessary precautions in handling laboratory reagents.
- The calibrators' values are lot-specific with the matched set listed in the Calibration Card.
- Calibration is recommended after each reagent lot change, special maintenance, or troubleshooting procedure.
- Do not mix calibrators from different lots.
- Please perform maintenance and standard operation as scheduled, including calibration and analysis to assure the performance of the measurement system.
- Recalibrate the system when controls are out of specified range.
- The calibrators were tested with CE-approved kits for antibodies to HIV-1, HIV-2, HCV, Syphilis TP and for HBsAg. All results are negative. However, as no testing method can rule out the potential risk of infection with absolute certainty, this material should be handled as potential biohazards.¹

- If there is any evidence of microbial contamination or excessive turbidity in the product, discard the vial.
- Material safety data sheet is available upon request.
- Disposal of all waste material should be in accordance with local guidelines.

Testing Method

- Refer to Mindray System Operation Manual and / or customer service to acquire information of calibration principle, use of calibrators, and validity of data of calibrators.
- Mix contents by gently inverting the vial no less than 10 times before use. Avoid bubble formation.
- Three levels of Anti-Tg calibrators are provided, which are 0, ~100 and ~1000 IU/mL. Test C0 three times and C1, C2 twice for calibration.
- Refer to calibration card for accurate information about lot-specific Anti-Tg concentration.

Preparation

- Take the vial out of refrigerator and let it sit at room temperature for no less than 30 min to reach room temperature.
- According to the calibration procedure described in the instrument manual, use the Calibrators' lot-specific value to set calibration parameters.

Storage and Stability

The calibrators are stable up to the expiration date as indicated on the label when stored in unopened vial at 2~8°C and protected from light. Once opened, it is stable for 30 days when refrigerated at 2~8°C.

Materials Required but not Provided

- Mindray CL-series Chemiluminescence Immunoassay Analyzer, reagent kits and controls.
- General laboratory equipment.

Traceability of Calibrator

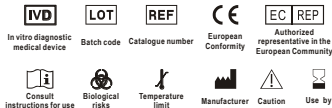
The traceability process is based on EN ISO 17511:2003,² the measurand (analyte) in the calibrators is traceable to the WHO International Standard Antibody to thyroglobulin 65/093. The concentration of calibrators is lot-specific on Mindray CL-series Chemiluminescence Immunoassay Analyzer. And accuracy of Mindray calibrators cannot be guaranteed, if other system or method is used, as bias may exist among different systems and methods. Details about traceability are available upon request.

Quality Control

Use Mindray controls as the samples to monitor the status of each calibration. The control result should be within the defined range as shown in the control value sheet. If the controls are out of specified range, the measurement system should be checked. For

instance, expiration date or storage condition of the calibrators, reagent and control; contamination of reagent; position of reagent or sample placed on the analyzer; the parameter setting in software; or performance of the analyzer. The assigned value of calibrators is fitted to Mindray CL-series Chemiluminescence Immunoassay Analyzer and is lot-specific.

Graphical Symbols

References

- HHS Publication, 5th ed., December 2009. Biosafety in Microbiological and Biomedical Laboratories.
- ISO 17511:2003. In vitro diagnostic medical devices—Measurement of quantities in biological samples—Metrological traceability of values assigned to calibrators and control materials.

© 2013 Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

All rights Reserved

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray Building, Keji 12th Road South, Hi-tech Industrial Park, Nanshan, Shenzhen, 518057 P.R.China

E-mail Address: service@mindray.com

Website: www.mindray.com

Tel: +86-755-26582888

Fax: +86-755-26582680

EC-Representative: Shanghai International Holding Corp. GmbH (Europe)

Address: Eifffestraße 80, Hamburg 20537, Germany

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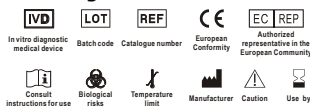
Simboli grafici

Riferimenti

1. HHS Publication, 5th ed., December 2009. Biosafety in Microbiological and Biomedical Laboratories.
2. ISO 17511:2003. In vitro diagnostic medical devices—Measurement of quantities in biological samples—Metrological traceability of values assigned to calibrators and control materials.

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Indirizzo: Mindray Building, Keji 12th Road
South, Hi-tech Industrial Park, Nanshan,
ShenZhen 518057, Cina.

Indirizzo e-mail: service@mindray.com

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Tel: +86-755-26582888

Fax: +86-755-26582680

Rappresentante CE: Shanghai
International Holding Corp. GmbH (Europa)
Indirizzo: Eiffestraße 80, Amburgo 20537,
Germania

Tel: 0049-40-2513175

Fax: 0049-40-255726

Anti-Tg Kalibratörleri

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Katalog No.	Ambalaj Boyutu
Anti-Tg211	C0:1×2,0 mL/flakon C1:1×2,0 mL/flakon C2:1×2,0 mL/flakon
Anti-Tg212	C0:1×1,2 mL/flakon C1:1×1,0 mL/flakon C2:1×1,0 mL/flakon

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Fabricant : Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Représentant EC : Shanghai International Holding Corp. GmbH (Europe)