Thyroid Function Multi Control

Order Information

Product	Catalog No.	Package Size
Thyroid Function Multi Control (L)	105-007383-00	1×5.0 mL/vial
	105-007379-00	3×5.0 mL/vial
	105-007371-00	6×5.0 mL/vial
Thyroid Function Multi Control (H)	105-007375-00	12×5.0mL/vial
	105-007384-00	1×5.0 mL/vial
	105-007380-00	3×5.0 mL/vial
	105-007372-00	6×5.0 mL/vial
	105-007376-00	12×5.0mL/vial

Intended Use

Mindray Thyroid Function Multi Control is used for quality control by monitoring the accuracy and precision of Mindray CL-series Chemiluminescence Immunoassay Analyzer and test ability of clinical laboratory in the quantitative measurement of thyroid function analytes.

Summary

Mindray CL measurement system is composed of Mindray CL-series Chemiluminescence Immunoassay Analyzer, Mindray reagent kits, calibrators, and controls. Mindray Thyroid Function Multi Control can be used to check whether QC result is within specified range or not. It is used with Mindray chemiluminescent immunoassays, including Free Triiodothyronine (CLIA), Free Thyroxine (CLIA), Total Triiodothyronine (CLIA), Total Thyroxine (CLIA), Thyroid-Stimulating Hormone (CLIA) and Thyroglobulin (CLIA).

Components

Thyroid Function Multi Control contains Free T3, Free T4, Total T3, Total T4, TSH and Tg analytes, with 0.09% sodium azide and 0.05% ProClin 300 as the preservative in human serum.

Warnings and Precautions

- For in vitro diagnostic use only. For laboratory professional use.
- Laboratory professionals must be trained to take all necessary precautions while handling laboratory reagents. Do not use reagents beyond the expiration date.
- The control values are lot-specific with the matched set listed in the value sheet.
- The controls should be run daily in parallel with the patient samples, after each calibration, or after the reagent lot has been changed.
- Please perform maintenance and standard operation as scheduled, including calibration and system analysis to assure the performance of measurement.
- The controls were tested with CE-approved kits for HIV-1/2 Antibody, HCV Antibody, Treponema pallidum Antibody and HBsAg. The results are all negative. However, as no testing method can rule out the potential risk of infection with absolute certainty, this material should be handled as potential biohazard. ¹
- If there is any evidence of microbial

- contamination or excessive turbidity in the product, contact Mindray Customer Service for help.
- Material safety data sheet is available upon request.
 - Disposal of all waste material should be in accordance with local guidelines.
 - The target values and ranges of Mindray Thyroid Function Multi Control are for reference only. Each laboratory can establish its own internal quality control procedure and set up its own target values and ranges.
 - Please confirm the integrity of the package before use. Do not use the reagents with damaged packages. The results can't be assured when the reagents are stored at inappropriate condition.
 - If the reagents are opened unintentionally before use, they shall be tightly capped and stored at 2-8°C, and then used within in-use stability time.
 - Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the local competent authority.
 - This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

Warning

H317 May cause an allergic skin reaction.
H412 Harmful to aquatic life with long lasting effects.

Prevention:

P261 Avoid breathing dust/fume/ gas/mist/vapours/spray.
P272 Contaminated work clothing should not be allowed out of the workplace.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P273 Avoid release to the environment.

Response:

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.
P302 + P352 IF ON SKIN: Wash with plenty of water.
P362 + P364 Take off contaminated clothing and wash it before reuse.

Disposal:

P501 Dispose of contents/ container in accordance with local regulation.

Test Method

Thyroid function multi controls have similar components as tested samples. Therefore, the controls can be tested by following the same procedure as tested samples, the target values and ranges are then calculated and compared to see if the controls meet specifications.

Preparation

- Refer to Mindray System Operation Manual and / or Customer Service to acquire information about control principle, control use, and control validity evaluation.
- Take the vial out of fridge, and let it stand for

no less than 30 minutes at room temperature.

- Mix contents by gently inverting the vial at least 10 times before use. Avoid bubble formation.
- Dispense the required volume into a sample cup, and then run QC test you have requested or have it analyzed in the same way as patient samples.

Storage and Stability

Thyroid Function Multi Control is stable up to the expiration date as indicated on the label when stored in unopened vial at 2-8°C. Once opened, it is stable for 30 days at 2-8°C or 60 days at -20°C.

Materials Required but not Provided

- Mindray CL-series Chemiluminescence Immunoassay Analyzer, reagent kits, and matched calibrators.
- General laboratory equipment.

Assay Values

The control values (the target value and range) determined by Mindray routine measurement procedure are listed in the target value sheet. The target value was obtained from Mindray measurement system and the range was calculated as the target values ± 3 standard deviations. The control values are lot-specific. Please check the lot number of each control before use.

Quality Control

Use Mindray controls as the samples to monitor the performance of system. The values of controls are lot-specific on Mindray CL-series Chemiluminescence Immunoassay Analyzer. The control result should be within the defined range as shown in the control target value sheet. If the controls are out of specified range, the measurement system and reagents should be checked, for instance, expiration date or storage condition of the calibrators, reagent and controls; possible contamination of reagent; position of reagent, sample placement on the analyzer; the parameter setting in software; or operation status of the analyzer. Each laboratory should establish its own internal quality control scheme and procedure for corrective and preventive actions.

Graphical Symbols

In vitro diagnostic medical device

Authorized representative in the European Community

Consult instructions for use

European Conformity

Catalogue number

Temperature limit

Manufacturer

Use-by date

Biological risks

Batch code

This way up

Control

Unique device identifier

Caution

References

- HHS Publication, 5th ed., December 2009. Biosafety in Microbiological and Biomedical Laboratories.

© 2022-2024 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights Reserved

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China

E-mail Address: service@mindray.com

Website: www.mindray.com

Tel: +86-755-81888998

Fax: +86-755-26582680

EC-Representative: Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestraße 80, Hamburg 20537, Germany

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© 2022-2024 Shenzhen Mindray Bio-Medical Electronics Co. Ltd. Tous droits réservés.
Fabricant : Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Représentant en Europe : Shanghai International Holding Corp. GmbH (Europe)