IL-6 Calibrators

Order Information

Catalog No.	Package Size
105-027173-00	C0: 4x0.35 mL/vial C1: 4x0.30 mL/vial C2: 4x0.30 mL/vial

Intended Use

Mindray IL-6 Calibrators (IL-6 CAL) are intended to calibrate the quantitative Interleukin-6 (IL-6) assay on Mindray CL-series Chemiluminescence Immunoassay Analyzer.

Summary

Mindray CL measurement system is composed of Mindray CL-series Chemiluminescence Immunoassay Analyzer, Mindray reagent kits, calibrators, and controls. IL-6 Calibrators contain Interleukin-6 (IL-6). The mathematical relationship between the measured responses and the known analyte concentration establishes the calibration curve. It is used to convert Relative Light Units (RLUs) measured from samples to quantitative analyte concentration.

Components

C0,C1,C2	IL-6 in buffered bovine serum albumin (BSA) at three concentration levels with 0.05% ProClin 300 and 0.09% sodium azide as preservatives;
----------	---

Warnings and Precautions

- For in vitro diagnostic use only. For laboratory professional use.
- Laboratory professionals must be trained to take all necessary precautions while handling laboratory reagents. Do not use reagents beyond the expiration date.
- The calibrators' values are lot-specific with the matched set listed in the Calibration Card.
- Calibration is recommended after each reagent lot change, special maintenance, or troubleshooting procedure.
- Do not mix calibrators from different lots.
- Please perform maintenance and standard operation as scheduled, including calibration and analysis to assure the performance of the measurement system.
- Recalibrate the system when controls are out of specified range.
- The calibrators were tested with CE-approved kits for HIV-1/2 Antibody, HCV Antibody, Treponema pallidum Antibody and HBsAg. All results are negative. However, as no testing method can rule out the potential risk of infection with absolute certainty, this material should be handled as potential biohazards.¹
- If there is any evidence of microbial

contamination or excessive turbidity in the product, discard the vial.

- Material safety data sheet is available upon request.
- Disposal of all waste material should be in accordance with local guidelines.
- Please confirm the integrity of the package before use. Do not use the calibrators with damaged packages. Inappropriate storage will lead to incorrect results.
- If the calibrators are opened unintentionally before use, they shall be tightly capped and stored at 2-8°C, and then used within in-use stability time.
- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the local competent authority.

Testing Method

- Refer to Mindray Operator's Manual and / or customer service to acquire information of calibration principle, use of calibrators, and validity of data of calibrators.
- Mix contents by gently inverting the vial no less than 10 times before use. Avoid bubble formation.
- Because of possible evaporation effects, the calibrators on the analyzers should be measured as soon as possible. One vial of calibrator can be used only once.
- Three levels of IL-6 calibrators are provided, which are 0, ~100 and ~1000 pg/mL. Test C0 three times and C1, C2 twice for calibration.
- Refer to calibration card for accurate information about lot-specific IL-6 concentration.

Preparation

- Take the vial out of refrigerator and let it stand for no less than 30 minutes at room temperature.
- According to the calibration procedure described in the instrument manual, use the Calibrators' lot-specific values to set calibration parameters.

Storage and Stability

The calibrators are stable up to the expiration date as indicated on the label when stored in unopened vial at 2-8°C. Once opened, it is stable for 30 days at 2-8°C or 90 days at -20°C.

Materials Required but not Provided

- Mindray CL-series Chemiluminescence Immunoassay Analyzer, reagent kits and controls.
- General laboratory equipment.

Traceability of Calibrator

The traceability process is based on EN ISO 17511:2020,² the measurand (analyte) in the

calibrators is traceable to the WHO International Standard Reference Reagent (IL-6 IS 89/548). The concentrations of calibrators are lot-specific on Mindray CL-series Chemiluminescence Immunoassay Analyzer, and cannot be used on the other system or method, as bias may exist among different systems and methods. Details about traceability are available upon request.

Quality Control

Use Mindray controls as the samples to monitor the performance of system. The control result should be within the defined range as shown in the control value sheet. If the controls are out of specified range, the measurement system should be checked. For instance, expiration date or storage condition of the calibrators, reagent and control; contamination of reagent; position of reagent or sample placed on the analyzer; the parameter setting in software; or performance of the analyzer. The assigned values of controls are fitted to Mindray CL-series Chemiluminescence Immunoassay Analyzer and are lot-specific.

Graphical Symbols

In vitro diagnostic medical device

Authorized representative in the European Community

Consult instructions for use

Catalogue number

Temperature limit

Manufacturer

Biological risks

Batch code

This way up

Caution

European Conformity

Use-by date

Unique device identifier

All rights Reserved.

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China

E-mail Address: service@mindray.com

Website: www.mindray.com

Tel: +86-755-81888998

Fax: +86-755-26582680

EC-Representative: Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestraße 80, 20537 Hamburg, Germany

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Identificador exclusivo do dispositivo

Referências

- HHS Publication, 5th ed., December 2009. Biosafety in Microbiological and Biomedical Laboratories.
- ISO 17511: 2020 In vitro diagnostic medical devices — Requirements for establishing metrological traceability of values assigned to calibrators, trueness control materials and human samples.

© 2022-2026 Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

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Fabricante: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Endereço: Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China

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Fabricant : Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Représentant en Europe : Shanghai International Holding Corp. GmbH (Europe)